



Type 6 Self-Adhesive Indicators

STOP

be sure...be ParaSure



Unprocessed indicator
Ready for use



PASS
Items in tray/pack
can be safely used



PASS
Items in tray/pack
can be safely used

Successfully processed indicators
Colour has changed to teal or darker



FAIL
Do NOT use items
in tray/pack



FAIL
Do NOT use items
in tray/pack



FAIL
Do NOT use items
in tray/pack

Unsuccessfully processed indicators
Evidence of original purple colour

PS1210 3.5 minutes at 134°C
PS1220 4.0 minutes at 134°C

PS1230 5.3 minutes at 134°C
PS1240 7.0 minutes at 134°C