



# PARASURE

## Standard Lumen PCD

- Unique simulation of cannulated instruments
- Conforms to recommendations in EN ISO/TS 15883-5
- Realistically challenges the process
- Easy to interpret results
- Polypropylene head
- Cost effective



The ParaSure Standard Lumen Process Challenge Device (PCD) simulates the cannulated devices used in Minimally Invasive Surgery (MIS), to effectively test the machine's cleaning performance.

The polypropylene head houses the indicator and is connected to a 30 centimetre length of PTFE tubing which terminates in a luer lock. Manufactured in accordance with EN ISO/TS 15883-5:2005, the Standard Lumen PCD challenges the machine to ensure effective soil removal by simulating the conditions found within cannulated instruments. The Standard Lumen PCD is designed to be used with the ParaSure Cleaning Efficacy Indicators (CEI), PS1600.

### Order Details

#### PS1620P

Standard Lumen  
PCD  
each



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)

### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777

Ref: ParaSure Tech Sheets 02/13 Rev.D



# PARASURE

## Standard Lumen PCD

### TECHNICAL SPECIFICATIONS

<b>Product name:</b>	Standard Lumen PCD
<b>Unit of measure:</b>	Each
<b>Conforms to:</b>	EN ISO/TS 15883-5:2005
<b>Materials:</b>	Polypropylene and PTFE
<b>Frequency of use:</b>	Every cycle
<b>Number of uses:</b>	Reusable
<b>Indicator:</b>	ParaSure Cleaning Efficacy Indicators (PS1600)
<b>Supplied with:</b>	Instructions for Use & Assessment Aid

### Assessment - Aid

#### Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
<b>INSUFFICIENT CLEANING PROCESS:</b>		
	Spot of test soil completely visible	Do not release batch!  Check load, temperature, dosage and time.  Repeat test.  In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
<b>SUCCESSFUL CLEANING PROCESS:</b>		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:  
[www.parasure.co.uk](http://www.parasure.co.uk)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777

[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)

[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)