



PARASURE

Single Lumen Channel Control 2

- Unique simulation of endoscope
- Conforms to recommendations in EN ISO/TS 15883-5
- Realistically challenges the process
- Easy to interpret results
- Polypropylene head
- 2mm internal diameter
- Cost effective



The ParaSure Single Lumen Channel Control 2 (SLCC2) simulates fine lumen devices and endoscope channels with a 2mm internal diameter, to effectively test the washer-disinfector's performance. The polypropylene head houses the indicator and is connected to a 1.5 metre length of PTFE tubing which terminates in a luer lock.

Manufactured in accordance with EN ISO/TS 15883-5:2005, the SLCC2

challenges the machine to ensure effective soil removal by simulating the conditions found within the endoscope.

The SLCC2 is designed to be used with the ParaSure Cleaning Efficacy Indicators (CEI), PS1600.

Order Details

PS1640P

Single Lumen
Channel Control 2
each



WWW.PARASURE.CO.UK



WWW.PARASUREORGANICS.COM

Registered Address:

ParaSure Limited Units B10/11 Oakendene
Estate Bolney Rd Cowfold West Sussex RH13
8AZ England

+44 (0)1403 864777

Ref: ParaSure Tech Sheets 02/13 Rev.D









PARASURE

Single Lumen Channel Control 2

TECHNICAL SPECIFICATIONS

| | |
|--------------------------|--|
| Product name: | Single Lumen Channel Control 2 |
| Unit of measure: | Each |
| Conforms to: | EN ISO/TS 15883-5:2005 |
| Materials: | Polypropylene and PTFE |
| Frequency of use: | Every cycle |
| Number of uses: | Reusable |
| Indicator: | ParaSure Cleaning Efficacy Indicators (PS1600) |
| Supplied with: | Instructions for Use & Assessment Aid |

Assessment - Aid

| Result | Assessment | Action needed |
|---|--|--|
|  | Reference indicator: Spot of test soil completely visible | |
| INSUFFICIENT CLEANING PROCESS: | | |
|  | Spot of test soil completely visible | Do not release batch! Check load, temperature, dosage and time. Repeat test. In case of repeated insufficient cleaning process: Call service technician. |
|  | Spot of test soil visible | |
|  | Spot of test soil visible | |
|  | Spot of test soil slightly visible | |
| SUCCESSFUL CLEANING PROCESS: | | |
|  | Spot of test soil completely removed | Release batch. No further action needed. |

For further information about any of our products, please visit:
www.parasure.co.uk



WWW.PARASURE.CO.UK



WWW.PARASUREORGANICS.COM

Registered Address:

ParaSure Limited Units B10/11 Oakendene
Estate Bolney Rd Cowfold West Sussex RH13
8AZ England

+44 (0)1403 864777