



# PARASURE

## Multi Lumen Channel Control

- Unique simulation of endoscope
- Conforms to recommendations in EN ISO/TS 15883-5
- Realistically challenges the process
- Easy to interpret results
- Polypropylene heads
- Cost effective



The ParaSure Multi Lumen Channel Control (MLCC) simulates an endoscope with 3 channels (1 x 1mm diameter and 2 x 2mm diameter), providing a unique solution to realistically test the effectiveness of the washer-disinfector. The MLCC combines 3 polypropylene head indicator holders, each of which is connected to a 1.5 metre length of PTFE tubing that terminates in a luer lock.

Manufactured in accordance with EN ISO/TS 15883-5:2005 (Annex R6 Test Pieces UK), the MLCC challenges the machine to ensure effective soil removal by simulating the conditions found within the endoscope.

The MLCC is designed to be used with the ParaSure Cleaning Efficacy Indicators (CEI), PS1600.

### Order Details

#### PS1650P

Multi Lumen  
Channel Control  
each



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)

### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777

Ref: ParaSure Tech Sheets 02/13 Rev.D



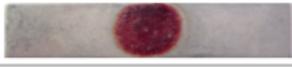
# PARASURE

## Multi Lumen Channel Control

### TECHNICAL SPECIFICATIONS

<b>Product name:</b>	Multi Lumen Channel Control
<b>Unit of measure:</b>	Each
<b>Conforms to:</b>	EN ISO/TS 15883-5:2005 (Annex R6 Test Pieces UK)
<b>Materials:</b>	Polypropylene and PTFE
<b>Frequency of use:</b>	Every cycle
<b>Number of uses:</b>	Reusable
<b>Indicator:</b>	ParaSure Cleaning Efficacy Indicators (PS1600)
<b>Supplied with:</b>	Instructions for Use & Assessment Aid

### Assessment - Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
<b>INSUFFICIENT CLEANING PROCESS:</b>		
	Spot of test soil completely visible	Do not release batch!  Check load, temperature, dosage and time.  Repeat test.  In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
<b>SUCCESSFUL CLEANING PROCESS:</b>		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:  
[www.parasure.co.uk](http://www.parasure.co.uk)



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777