



Steam Sterilization Load Controls




be sure...be ParaSure


 **3.5 min. at 134 °C** Sterilizing conditions turn indicator from light blue to dark. **LOT 121018** **PCN: PS1410** **STEAM** ©PARASURE **Unprocessed indicator**
Large square is pale blue

 **Successfully processed indicator**
Large square is as dark or darker than small square

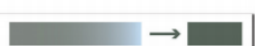
PASS
Items in tray/pack can be safely used

 **Unsuccessfully processed indicator**
Large square is paler than small square
Cause: only temperature present

FAIL
Do NOT use items in tray/pack

 **Unsuccessfully processed indicator**
Only partial colour change in large square
Cause: insufficient steam penetration

FAIL
Do NOT use items in tray/pack

 **Unsuccessfully processed indicator**
Evidence of original pale blue colour in large square
Cause: non-condensable gases present

FAIL
Do NOT use items in tray/pack

PS1400 EN 867-5 Helix Device
PS1410 3.5 minute Load Control
PS1420 4.0 minute Load Control

PS1430 5.3 minute Load Control
PS1440 7.0 minute Load Control
PS1450 18.0 minute Load Control