

## InstaClean

Ready to use neutral pre-cleaning kit for flexible endoscopes

- Integrated neutral solution and contoured sponge
- Neutral pH and non-corrosive
- Two stage process: wipe down, then flush through
- For use at bedside, post-procedure
- Single use: no re-using of soiled detergents
- Prevents drying of soils and coagulation
- Non-toxic
- CE marked



When reprocessing an endoscope, it is imperative that any soil or debris remaining either on or in the channels of the endoscope is not allowed to dry as this will become extremely difficult to remove. It is therefore now a general recommendation in numerous national and international guidelines that endoscopes are wiped down and flushed at the point of use before being transported for reprocessing.

The ParaSure InstaClean offers a single use, cost effective solution to this requirement, combining a contoured sponge and 200ml of pre-mixed neutral detergent. InstaClean provides a simple two-step process supplied in a single container. Firstly, the sponge is used to wipe down the insertion tube and then the remaining detergent is then used simply to flush through the endoscope.

The neutral pH means the detergent is non-corrosive to ensure safety for the user and greater longevity for the metal, rubber and plastic parts of the endoscope.

### Order Details

PS2300

InstaClean Neutral Pre-Clean Kit  
box of 50 bags

### TECHNICAL SPECIFICATIONS

Product name:	InstaClean Neutral Pre-Clean Kit
Unit of measure:	box of 50 bags
Shelf life:	2 years from date of manufacture
pH value:	7.0 to 7.5
Number of uses:	Single use

## InstaClean Instructions for Use



Open the pouch and remove the sponge, taking care to squeeze any excess fluid back into the pouch

Use the specially contoured sponge to wipe down the insertion tube and then discard the sponge

Place the distal end of the endoscope into the pouch and begin suctioning detergent into the tube. Alternate suctioning air and detergent for best results

For further information about any of our products, please visit:

[www.parasure.co.uk](http://www.parasure.co.uk)

ParaSure Limited

Office: Squirrels | Kithurst Park | Storrington | West Sussex | RH20 4JH | England  
Warehouse: Units B10/B11 | Oakendene Estate | Bolney Road | Cowfold | West Sussex | RH13 8AZ  
Telephone: +44 (0) 1903 741390 Email: [info@parasure.co.uk](mailto:info@parasure.co.uk)