

STOP!


be sure...be ParaSure

Type 5 Integrator Steam



Unprocessed indicator

The three additional triangles are pink

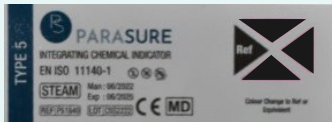


Successfully processed indicator

All three additional triangles are now black.

PASS

Items in tray/pack can be safely used

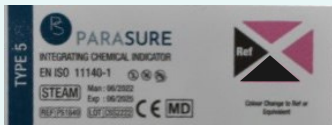

Unsuccessfully processed indicator

Only partial colour change to bottom triangle.
Cause: only temperature present

FAIL

Do NOT use items in tray/pack



Unsuccessfully processed indicator

Only partial colour change to top and middle triangle.
Cause: insufficient steam penetration

FAIL

Do NOT use items in tray/pack



Unsuccessfully processed indicator

Only partial colour change to middle and bottom triangle.
Cause: non-condensable gases present

FAIL

Do NOT use items in tray/pack

PS1840