




# PARASURE

## Synthetic Cleaning Efficacy Indicators

- Cost effective method for monitoring soil removal
- Conform to EN ISO 15883 1-2-5 and UNI/TR 111408
- Realistically challenges the process
- To be used in conjunction with the ParaSure Standard Basket PCD (PS1610)
- Easy to interpret results
- Instant results
- Non-toxic and latex-free

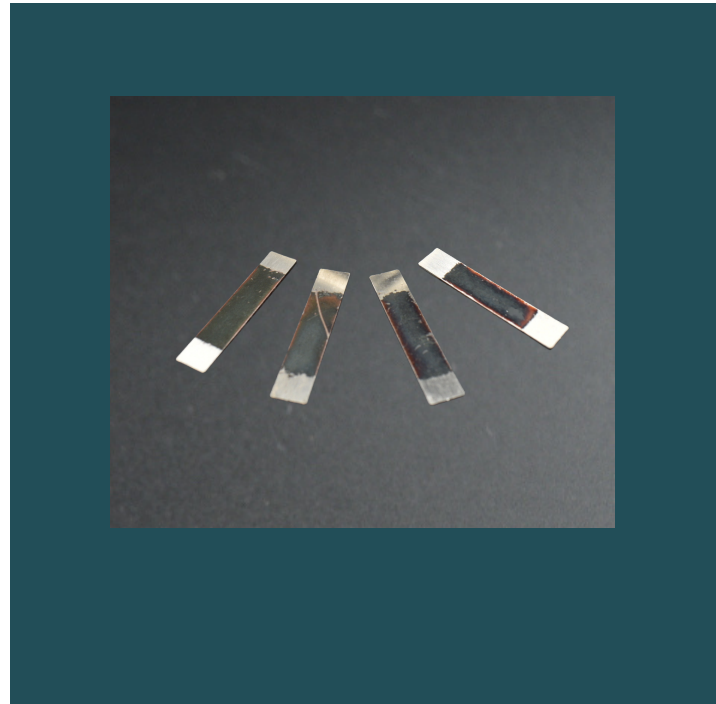

The ParaSure Synthetic Cleaning Efficacy Indicators offer a unique and cost-effective option for all healthcare facilities to ensure the decontamination process and soil removal efficacy is in accordance with EN ISO 15883 1-2-5. The indicators are designed to be used in conjunction with the ParaSure Standard Metal Basket (PS1610).

This system provides a cost-effective way of ensuring the washer-disinfector is functioning correctly by removing all soil from the indicator. The diagnostic technology gives a clear and instant result when used with the comprehensive reference guide (see overleaf).

### Order Details

#### PS1601

Synthetic Cleaning Efficacy Indicators

supplied as  
1 box of 100  
indicators



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777



# PARASURE

## Synthetic Cleaning Efficacy Indicators

### TECHNICAL SPECIFICATIONS

<b>Product name:</b>	Synthetic Cleaning Efficacy Indicators
<b>Unit of measure:</b>	1 box of 100 indicators
<b>Shelf life:</b>	2 years
<b>Conforms to:</b>	EN ISO 15883-5 and UNI/TR 111408
<b>Materials:</b>	Stainless steel
<b>Frequency of use:</b>	Every cycle
<b>Number of uses:</b>	Single use
<b>Supplied with:</b>	Instructions for Use & Assessment Aid

### Assessment Aid

Result	Assessment	Action needed
	Reference indicator: Test soil completely visible	
<b>INSUFFICIENT CLEANING PROCESS:</b>		
	Test soil completely visible	Do not release batch! Check load, temperature, dosage and time. Repeat test.  In case of repeated insufficient cleaning process: Call service technician.
	Test soil visible	
	Test soil slightly visible	
	Test soil slightly visible	
<b>SUCCESSFUL CLEANING PROCESS:</b>		
	Test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:  
[www.parasure.co.uk](http://www.parasure.co.uk)

#### Registered Address:

ParaSure Limited Units B10/11 Oakendene  
Estate Bolney Rd Cowfold West Sussex RH13  
8AZ England

+44 (0)1403 864777



[WWW.PARASURE.CO.UK](http://WWW.PARASURE.CO.UK)



[WWW.PARASUREORGANICS.COM](http://WWW.PARASUREORGANICS.COM)