





STOP! be sure...be ParaSure

Moving Line Type 5

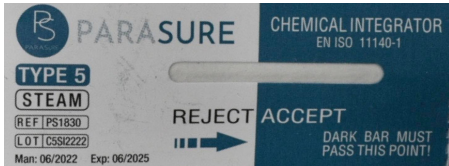


Unprocessed indicator
Blank bar.

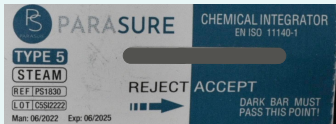

Successfully processed indicator

The colour change has gone beyond the accept line.


PASS
Items in tray/pack can be safely used

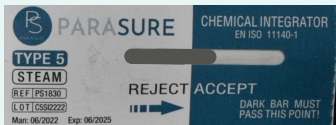

Successfully processed indicator

The colour change has gone beyond the accept line.


PASS
Items in tray/pack can be safely used

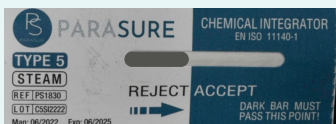

Unsuccessfully processed indicator

The colour change has not reached beyond the accept line.
Cause: insufficient steam penetration

FAIL
Do NOT use items in tray/pack

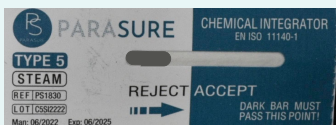

Unsuccessfully processed indicator

The colour change has not reached beyond the accept line.
Cause: non-condensable gases present

FAIL
Do NOT use items in tray/pack

PS1830

