



PARASURE

Single Lumen Channel Control 1

- Unique simulation of endoscope
- Conforms to recommendations in EN ISO/TS 15883-5
- Realistically challenges the process
- Easy to interpret results
- Orange polypropylene head
- 1mm internal diameter
- Cost effective



The ParaSure Single Lumen Channel Control 1 (SLCC1) simulates fine lumen devices and endoscope channels with a 1mm internal diameter, to effectively test the washer-disinfector's performance. The polypropylene head houses the indicator and is connected to a 1.5 metre length of PTFE tubing which terminates in a luer lock.

Manufactured in accordance with EN ISO/TS 15883-5:2005, the SLCC1 challenges the machine to ensure effective soil removal by simulating the conditions found within the endoscope. The SLCC1 is designed to be used with the ParaSure Cleaning Efficacy Indicators (CEI), PS1600. With its orange polypropylene head, the SLCC1 is easily distinguished from the SLCC2.

Order Details

PS1630P

Single Lumen
Channel Control
1 each

Registered Address:

ParaSure Limited Units B10/11 Oakendene
Estate Bolney Rd Cowfold West Sussex RH13
8AZ England

+44 (0)1403 864777



WWW.PARASURE.CO.UK



WWW.PARASUREORGANICS.COM








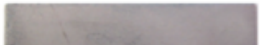
PARASURE

Single Lumen Channel Control 1

TECHNICAL SPECIFICATIONS

Product name:	Single Lumen Channel Control 1
Unit of measure:	Each
Conforms to:	EN ISO/TS 15883-5:2005
Materials:	Polypropylene and PTFE
Frequency of use:	Every cycle
Number of uses:	Reusable
Indicator:	ParaSure Cleaning Efficacy Indicators (PS1600)
Supplied with:	Instructions for Use & Assessment Aid

PARASURE Assessment-Aid

Result	Assessment	Action needed
	Reference indicator: Spot of test soil completely visible	
INSUFFICIENT CLEANING PROCESS:		
	Spot of test soil completely visible	Do not release batch! Check load, temperature, dosage and time. Repeat test. In case of repeated insufficient cleaning process: Call service technician.
	Spot of test soil visible	
	Spot of test soil visible	
	Spot of test soil slightly visible	
SUCCESSFUL CLEANING PROCESS:		
	Spot of test soil completely removed	Release batch. No further action needed.

For further information about any of our products, please visit:
www.parasure.co.uk



WWW.PARASURE.CO.UK

Registered Address:

ParaSure Limited Units B10/11 Oakendene
Estate Bolney Rd Cowfold West Sussex RH13
8AZ England



WWW.PARASUREORGANICS.COM

+44 (0)1403 864777